NECK **2** GO 

< timetorelax

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since
2000

< timetorelax

Since the company was founded back in the year 2000, the name CASADA has become synonymous with high-quality products in the wellness and fitness sector.

Today, CASADA devices are available worldwide. Continuous development, unmistakable design and a strict focus on improving quality of life are just as much characteristic of CASADA products as reliable and top-class quality.

Congratulations

By purchasing this massage device you have demonstrated an awareness for your health.

So that you can benefit from the advantages of this device for a long time, we would ask you to read and observe the safety instructions carefully.

We wish you great joy with your personal Neck2Go.

Table of contents

Safety instructions	12
Product description.....	13
Technical data	13
Charging	13
Control panel	14
Usage.....	14
Specials.....	15
Troubleshooting	16
Care Maintenance	16
EU Declaration of Conformity	17
Warranty conditions	17

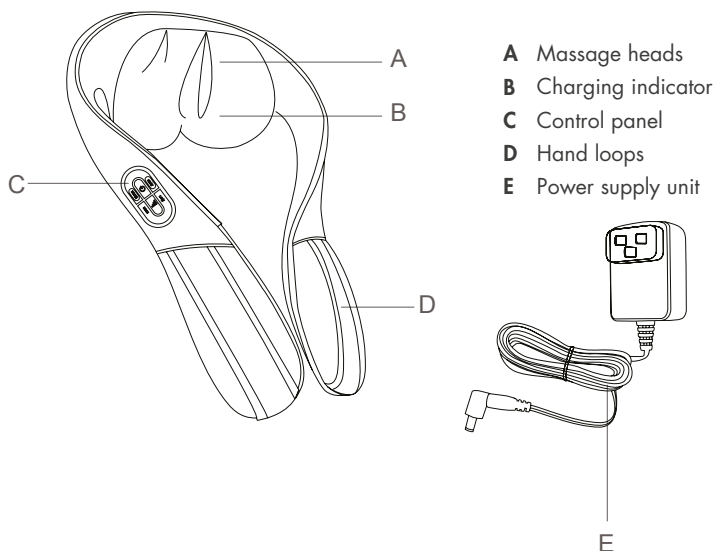
Safety instructions

Please read the following instructions carefully before putting your massage device into use to ensure trouble-free operation and optimum efficiency. Please retain these operating instructions for further use!

- The massage device conforms to recognized technical principles and the latest safety regulations.
- This device does not require any maintenance.
- Any possible repairs may only be carried out by authorized expert staff. Improper use and unauthorised repair are prohibited for safety reasons and lead to a loss of warranty.
- Never touch the power plug with wet hands.
- Please avoid contact of the device with water, high temperatures and direct sunlight.
- To avoid the danger of short-circuits, disconnect the power cord when not used for long periods of time.
- Do not use any damaged cables, plugs or loose sockets. If the plugs or cords are damaged, they must be replaced by the manufacturer, a service representative or by qualified personnel.
- In case of malfunction, disconnect immediately from the mains. Misuse or incorrect use excludes any liability for damage.
- In order to avoid excessively stimulating the muscles and nerves, the recommended massage time of 30 minutes should not be exceeded.
- Never use pointed or sharp objects.
- Each massage – even a hand massage – must be refrained from during pregnancy or if one or more of the following complaints are present in the massage area: recent injuries, thrombotic diseases, all kinds of inflammations and swellings, and cancer. It is recommended to consult the doctor before taking a massage for the treatment of illness or infirmity.
- Do not use this device while driving.
- If you are dependent on electrical aids e.g. pacemakers, please be sure to consult your doctor for medical advice before taking a massage.
- This appliance can be used, by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children are not allowed to play with the device.
- Children without supervision should, not do Cleaning and maintenance.
- Persons who are sensitive to heat shall be careful when using the unit.
- Use the device only with the supplied equipment.

Product description

Technical data

Product measurements (L x H x W) 360 x 250 x 220 mm

Weight 2,05 kg

Power source
input 100-240V ~ 50/60Hz
output DC 12.8 V, 1800 mA

Rated power 23 W

Automatic runtime 10 minutes

Certificates

Charging

1. Connect the power supply unit to the product and to the power socket.
2. During charging, the charging indicator lights on the massage heads will light up (green).
3. When the battery is fully charged, the green lights will go out.
4. After charging, please unplug the power supply unit from the power socket and disconnect it from the product.

Control panel

1. To start the device, press and hold the POWER  button. The device will start operating on mode 1 at low speed. To switch it off, press the POWER button again.
2. MODE  button
The device will start on mode 1 (green light: massage heads knead inwards). Press the button to change to mode 2 (red light: massage heads knead outwards). Press the button again to change to mode 3 (yellow light: massage heads knead inwards and outwards).
3. The device has 2 speed settings. It automatically starts on a low speed. Press the SPEED  button to increase the massage speed or to lower it again.
4. Press this button to start or stop the HEAT FUNCTION .

The automatic massage programme lasts for 10 minutes. The massage device will automatically switch itself off after this time to prevent overheating. Unplug the power cable from the socket after every use.

Usage

Place the Neck2Go on the back of the neck and let the loops fall forward over your chest. Reach into the loops on the left and right and position the massage device on your neck so that it feels comfortable (Fig. 1).

The device can alternatively be used to massage the thighs (Fig. 2).

(Fig. 1)

(Fig. 2)

Specials

KNEADING

Skin and muscles are either clasped and kneaded between the thumb and the index finger or with both hands. This massage technique is used in particular to ease tension.

HEATING

A deep penetrating and heat-radiating effect in which one can perfectly relax. This function is particularly beneficial for muscle tension due to the deep reaching effect.

REVERSE

The reverse function changes the direction of the massage heads.

BATTERY

Battery-powered massage and wellness experience.

SPEED

Massage speed is adjustable.

WIRELESS

Wireless massage device.

THUMB PRESSURE

Round massageheads, based on human thumb tips, grant a very delicate massage experience.

Troubleshooting

The following table provides a summary of the most common problems. If you are unable to solve the issue using the information below, please get in contact with the manufacturer or an authorised dealer immediately.

Fault	Possible cause	Remedial action
The device is not working.	Low battery level.	Charge the device.
	The device switches itself off after the automatic operating time of 10 minutes.	Press the POWER button to restart the massage.
	You may have used the product for 40 minutes or more, which has triggered the overheat protection.	Let the product cool down for 60 minutes before using it again.
	The device has been subjected to excessive strain.	Reduce the strain on the device and start it again.
The device seems hotter than usual.	The device has been overloaded.	Let the product cool down before using it again.
The device is making loud noises.	The noises are caused by the mechanical components in the device.	This is normal and does not mean that the product is damaged.

Care | Maintenance

CLEANING

1. Do not use steel wool, cleaning agents or corrosive liquids (such as benzine and acetone) to clean the product.
2. Disconnect the plug from the power socket.
3. Clean the product with a soft, dry cloth.
4. Keep liquids and naked flames away from the device.

STORAGE

1. Disconnect the device from the power supply when not in use.
2. Store the product in a cool, dry place away from sunlight.
3. Do not subject the device to heavy weight.

If the cover is damaged or a mechanical fault occurs, please stop using the product and contact the manufacturer or an authorised dealer immediately.

EU Declaration of Conformity

Complies with European Directives EMC 2014/30/EU, LVD 2014/35/EU and RoHs.

Warranty conditions

The retailer will provide a statutory period of guarantee for the product described on the reverse. Warranty is valid from the date of purchase. The purchase date must be documented with proof of purchase. The retailer will repair or replace products that are registered within Germany in which a defect has been found free of charge. This does not include parts subject to wear such as fabric covers. The requirement is that the defect device is sent in with proof of purchase before expiration of the warranty period.

The warranty claim shall not be applicable if a fault is established which has arisen e.g. due to external influences or as a consequence of a repair or modification which was not carried out by the manufacturer or an authorized dealer. The warranty on the part of the retailer is limited to repair or exchange of the product. Within the framework of this warranty, the manufacturer or retailer will not assume any further liability and is not responsible for damages that have arisen as a result of non-compliance with the operating instructions and/or improper use of the product.

If upon examination of the product by the manufacturer it is found that the warranty request does not concern a defect covered by the warranty or that the warranty period has expired, the customer must bear the costs for the inspection and repair.

Legal note: All contents do not contain healing statements. The diagnosis and treatment of illnesses and other physical disorders require a treatment by a physician, non-medical practitioner or therapist. The statements are strictly informative and may not be used as a substitute for a professional medical treatment. Every user is beholden to ascertain by accurate examination and, if required, after consultation of a specialist, if the application is the right one for their specific cases. Each dose or application takes place at the risk of the user. We dissociate ourselves from any healing statements or promises.

Casada International GmbH
Obermeiers Feld 3
33104 Paderborn, Germany
info@casada.com | www.casada.com

Copyright Images and text are subject to the copyright of CASADA International GmbH and may not be used without express consent.

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Copyright © 2022 CASADA International GmbH.
All rights reserved.
Copyright ©2022 Casada International GmbH. Todos los derechos reservados.
Copyright ©2022CASADA International GmbH. Все права защищены
Copyright © 2022 CASADA International GmbH. Minden jog fenntartva.